

GCX Mounting Installation Guide

M-Series Arm for Philips Anesthesia Accessory Track



CAUTION: Counterweights must be installed in the base of the Philips AX700 before mounting Arm.

<p>(X1)</p>	<p>(X1)</p>	<p>(X1)</p>	<p>(X2) 1/4-20 x 7/16" SHCS</p>
			<p>(X3) 10-32 x 3/8" SHCS</p>
			<p>(X3) 10-32 x 7/16" PHMS</p>
			<p>(X3) #10 x 5/8" PHSMS</p>
<p>M-Series Arm: Refer to separate installation guide.</p>			<p>(X1) 3/16" (X1) 5/32"</p>

(X1) 1/4-20 x 7/16" SHCS

4mm

3/16"

Warning: Torque 100 in-lbs 11.3 Nm

(X1) 1/4-20 x 7/16" SHCS

3/16"

