

Installation Guide


Power Supply Mounting Bracket for up to Three (3) Power Supply Holsters – Channel Mount

The purpose of this guide is to describe mounting of Bracket in channel and attachment of power supply holster to Bracket.

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):

Item #	Description	Qty
1	Power Supply Bracket	1
2	#10-32 x 5/16" Flat Head Machine Screw (FHMS), 100°	6
3	1/8" Hex Wrench	1

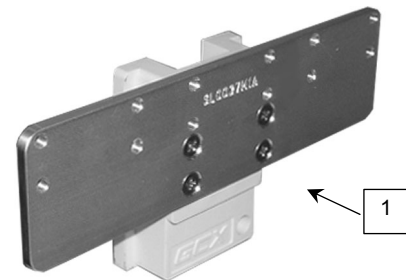

Tools Required

1/8" hex wrench (provided), Phillips screwdriver (not provided).

Mounting the Power Supply Mounting Bracket in the Channel

1. Insert the Channel Slide (rear of Power Supply Bracket) in the Channel and slide to mounting position. Using the 1/8" hex wrench provided, tighten set screw at bottom right corner of Channel Slide.



Attaching Power Supply Holsters to the Mounting Bracket

Six (6) #10-32 x 5/16" FHMS are provided for mounting up to three (3) Power Supply Holsters on the Mounting Bracket. Three (3) Holster-mounting positions are shown below left.

1. Fasten each Holster to the Mounting Bracket with two (2) #10-32 x 5/16" FHMS as shown below right.

