

Installation Guide Power Supply Cover

The purpose of this guide is to describe the installation of the Power Supply Cover.

Parts Reference

The following parts and hardware are included in this installation kit.

Item #	Description	Qty
1	Power Supply Cover	1
2	Adjustable Stop	1




Tools Required:


- Phillips Screwdriver (not provided)

Installing the Power Supply Cover

1. Slide the Adjustable Stop into top of channel and move to desired mounting position. Using a Phillips screwdriver, tighten Center Screw to secure position in channel.

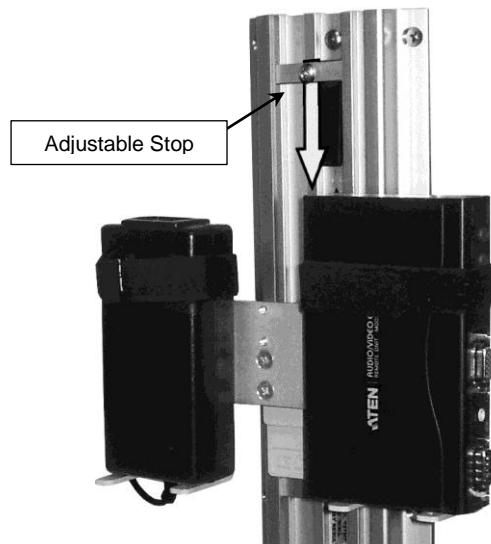

2. Guide the Channel Slide on the back of the Power Supply Cover into the top of the channel and lower the assembly down on to the Adjustable Stop.

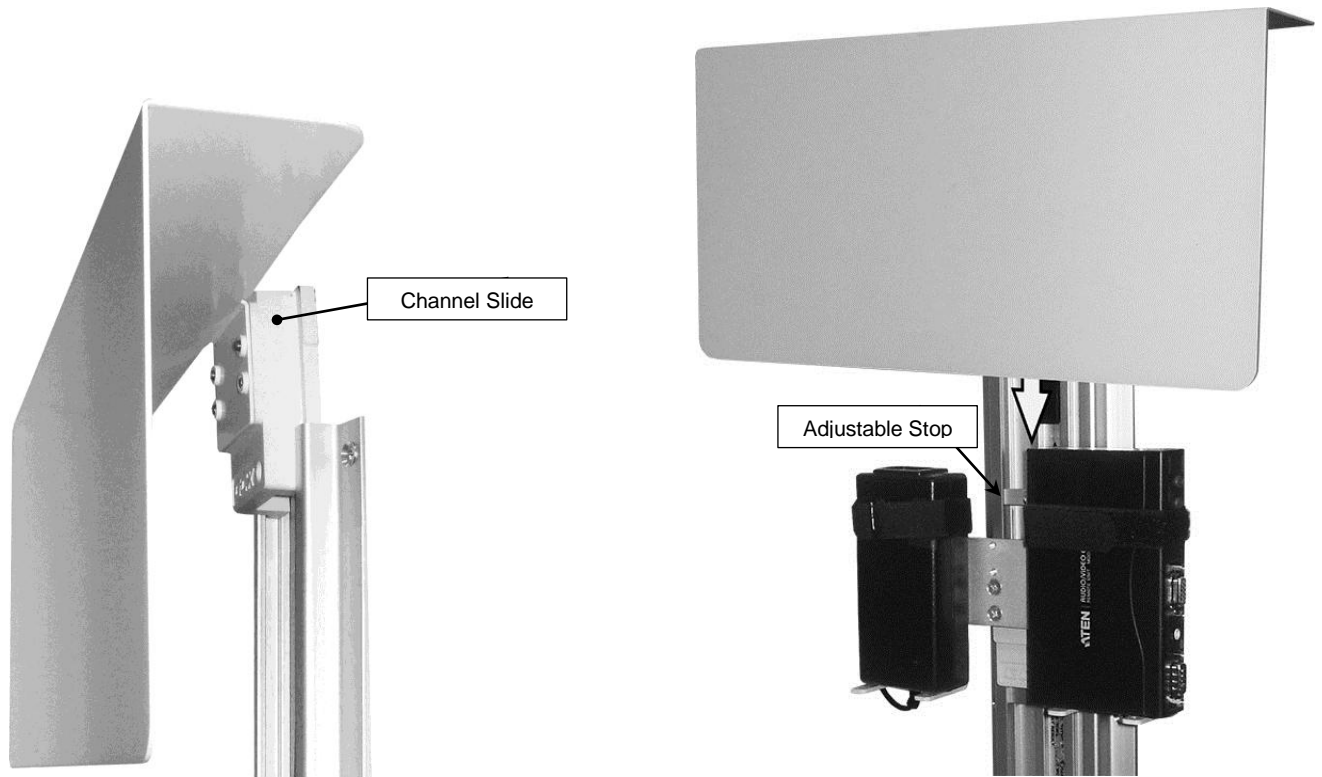

Periodic Maintenance

All fasteners associated with the mounting system should be inspected periodically and tightened as necessary.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Steel wool or other abrasive material should never be used.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the assembly immediately, using a water-dampened cloth. Dry the mounting assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.