



Installation Guide


E-Cylinder Channel Mounts & VHRC/RC Rear Accessory Track Mounts
(Cylinder Diameter: 3.5" - 4.5" / 89 - 114.3mm)
(Maximum Cylinder Weight – 20 lbs / 9.1kg Full)

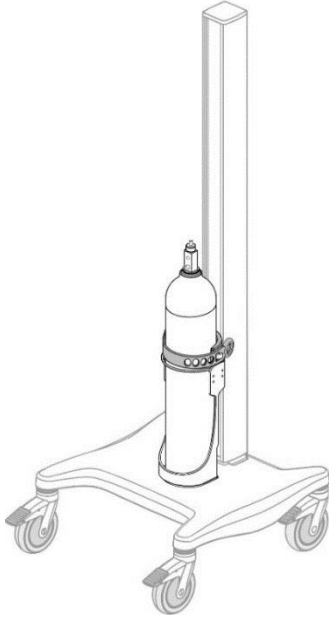


E-Cylinder Channel Mount
WM-0009-88

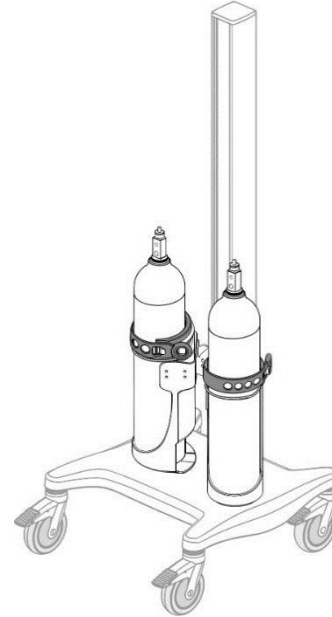

Dual E-Cylinder Channel Mount
WM-0009-89

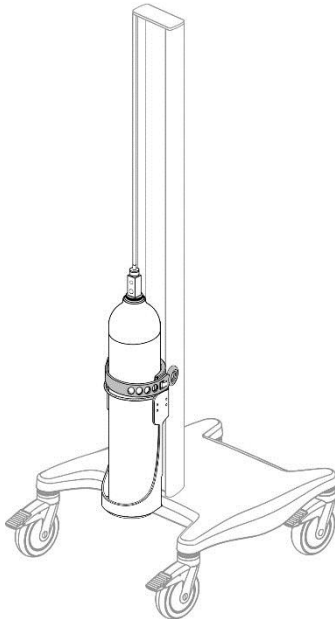

E-Cylinder VHRC/RC Rear Accessory Track Mount
RST-0008-88

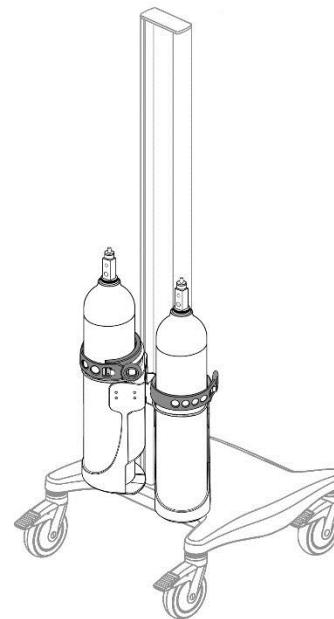

Dual E-Cylinder VHRC/RC Rear Accessory Track Mount
RST-0008-89






Safety Cable must be correctly secured when E-Cylinder is installed. Refer to page 6 for correct installation.

Parts Reference- E-Cylinder Channel Mount

Item #	Description	Qty
1	E-Cylinder Channel Mount	1
2	1/8" Hex Wrench	1




Installing the E-Cylinder Channel Mount

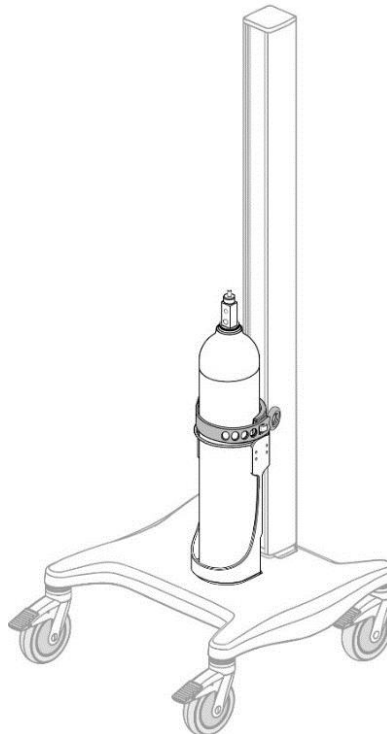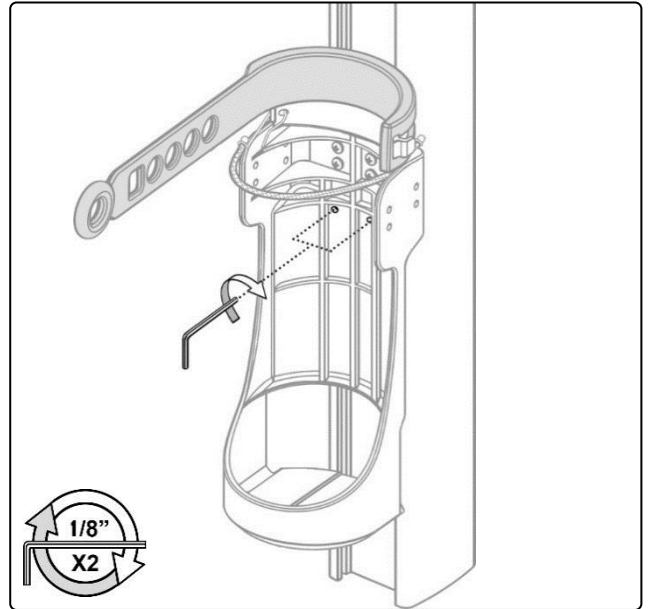
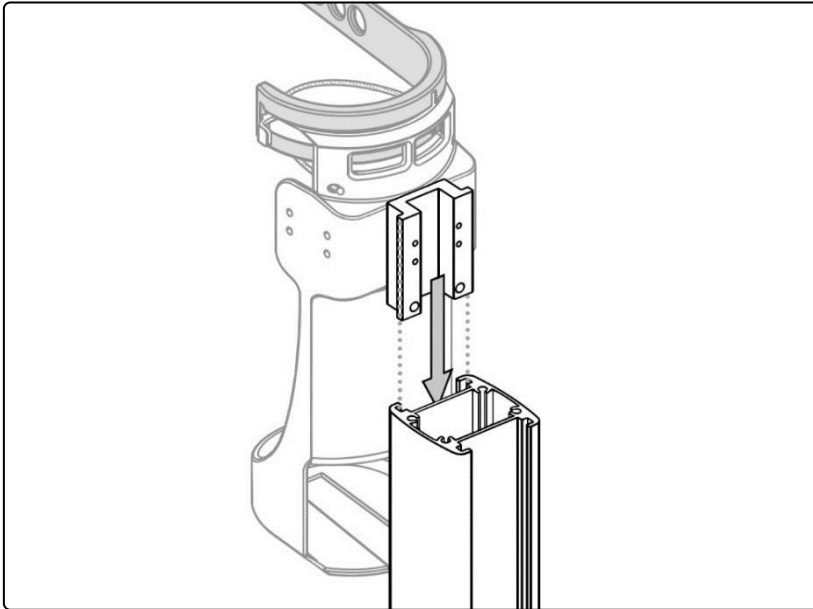


Parts Reference- Dual E-Cylinder Channel Mount

Item #	Description	Qty
1	Dual E-Cylinder Channel Mount	1
2	1/8" Hex Wrench	1

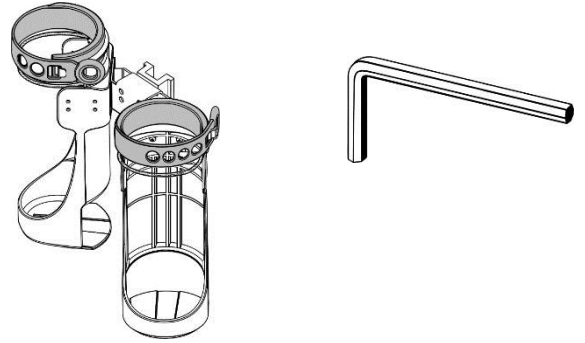


Installing the Dual E-Cylinder Channel Mount

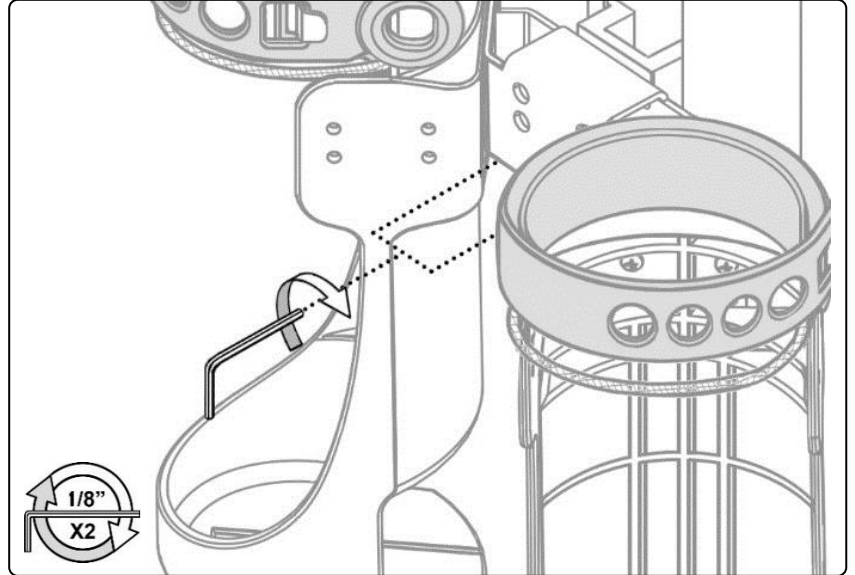
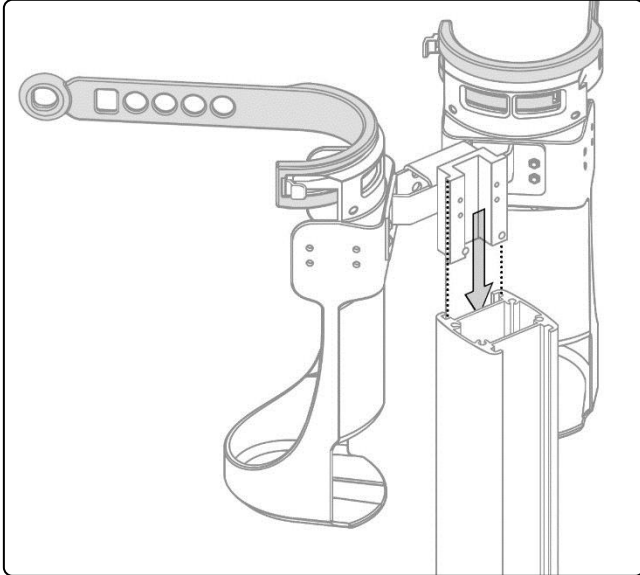


Parts Reference- E-Cylinder Rear Accessory Track Mount

Item #	Description	Qty
1	E-Cylinder Mount	1
2	Rear Accessory Track Mounting Plate	1
3	10-32 x 1/2" Pan Head Machine Screw (PHMS)	4
4	1/8" Hex Wrench	1

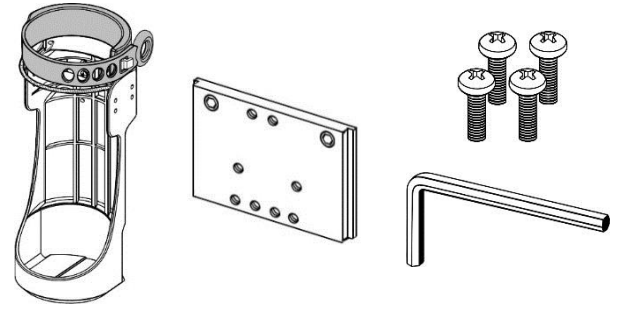


Installing the E-Cylinder Rear Accessory Track Mount


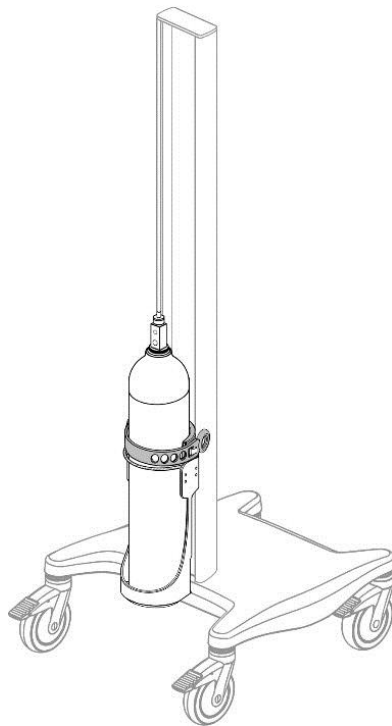
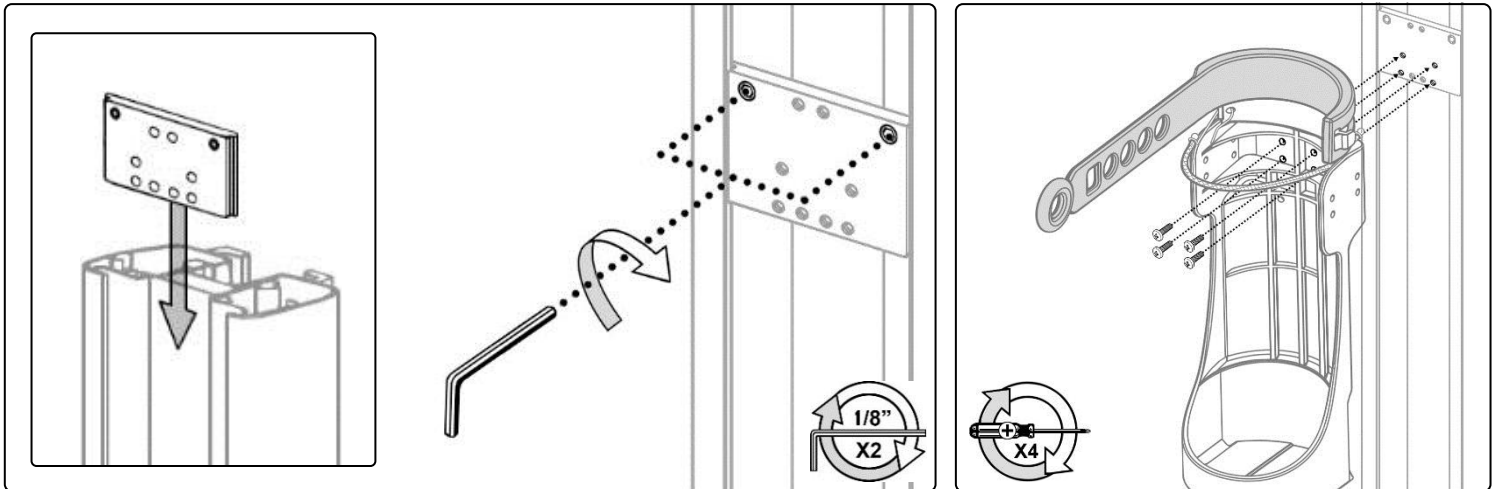


Parts Reference- Dual E-Cylinder Rear Accessory Track Mount

Item #	Description	Qty
1	Dual E-Cylinder Rear Accessory Track Mount	1
2	1/8" Hex Wrench	1

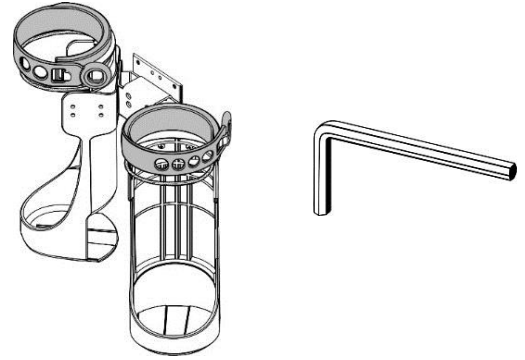




Installing the Dual E-Cylinder Rear Accessory Track Mount

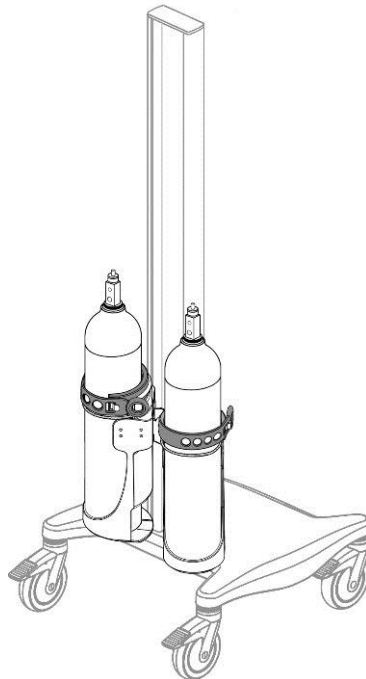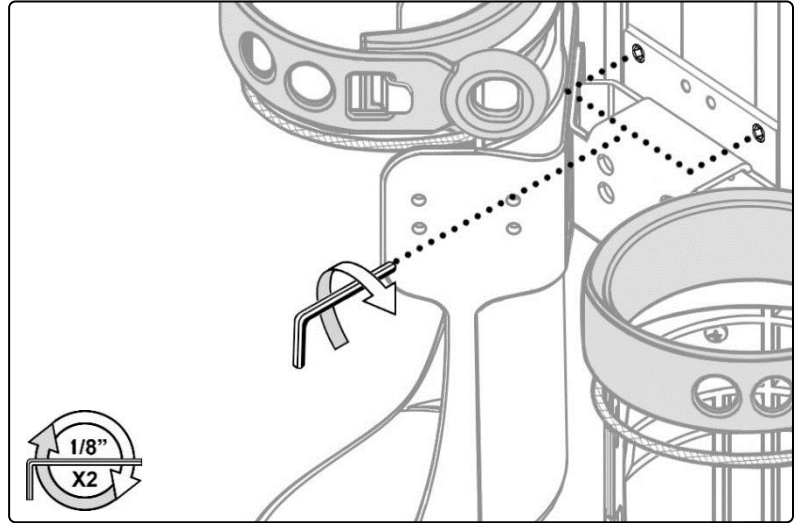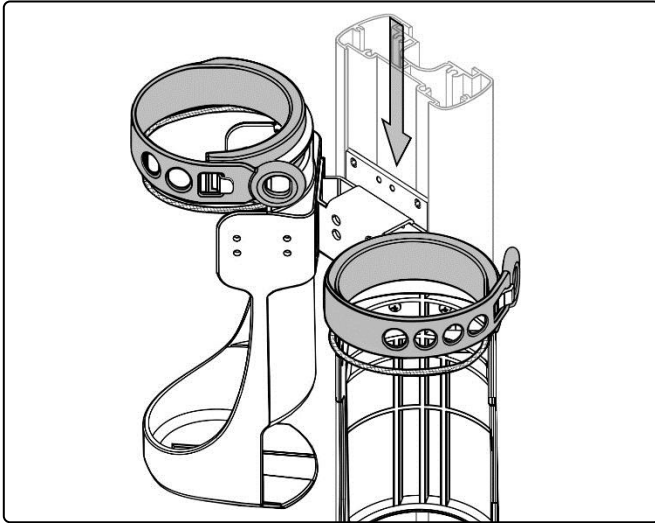



Safety Cable must be correctly secured when E-Cylinder is installed.

Securing Safety Cable

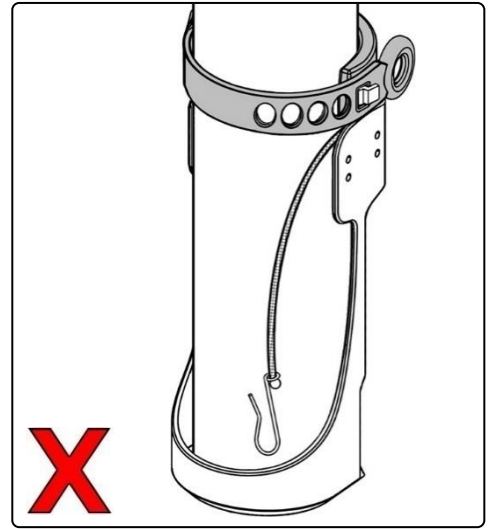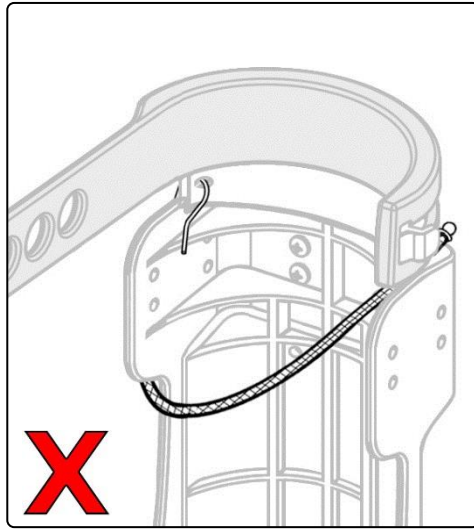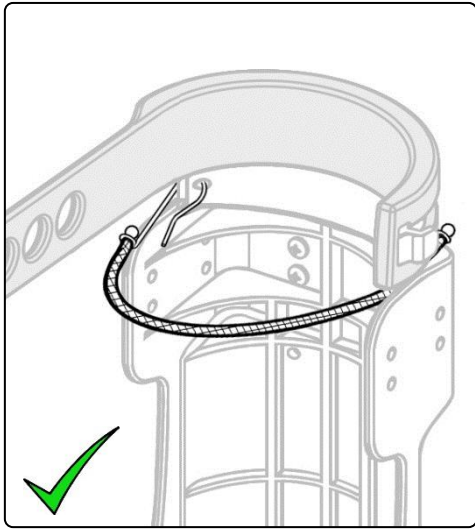

Installing E-Cylinder

