


Top Plate for GE Carestation 620/650 Guide

The purpose of this guide is to:

Describe the installation of 10"/25.4cm Channel to top plate.

Describe the installation of Carestation 600 Series monitor mount to top plate.


Describe the installation of top plate to Carestation 620/650.

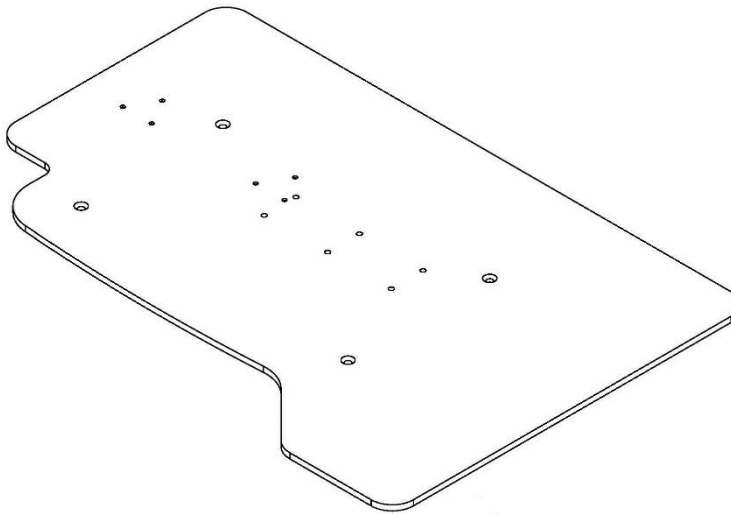

Parts Reference

The following parts and hardware are included in this installation kit:




Item #	Description	Qty.
1	Top Plate for Carestation 620/650	1
2	10"/25.4cm Channel	1
3	M5x10mm FHMS SS PHIL w/Patch	4
4	M6x30mm FHMS SS PHIL w/Patch	4
5	.375" Round Blank Label, Silver	9

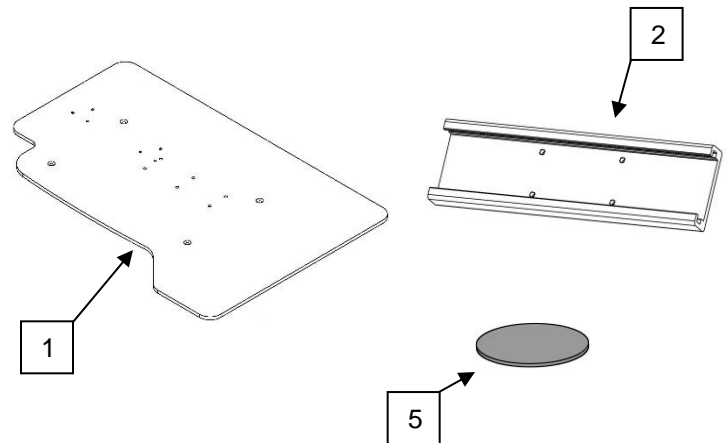



Tools Required:

- Phillips Screwdriver (Not provided)
- 5mm Hex Wrench (Not provided)

1

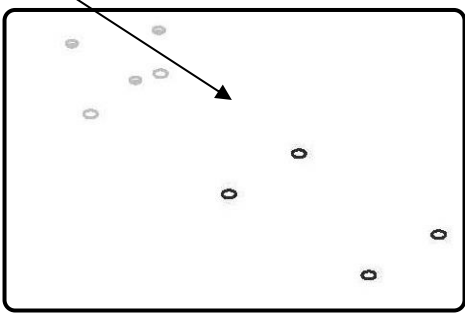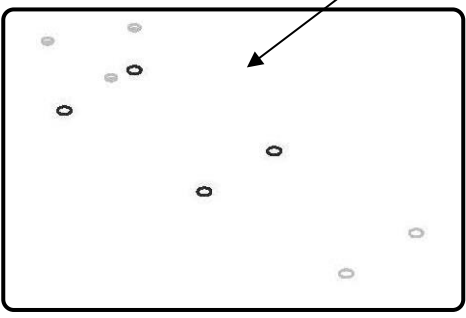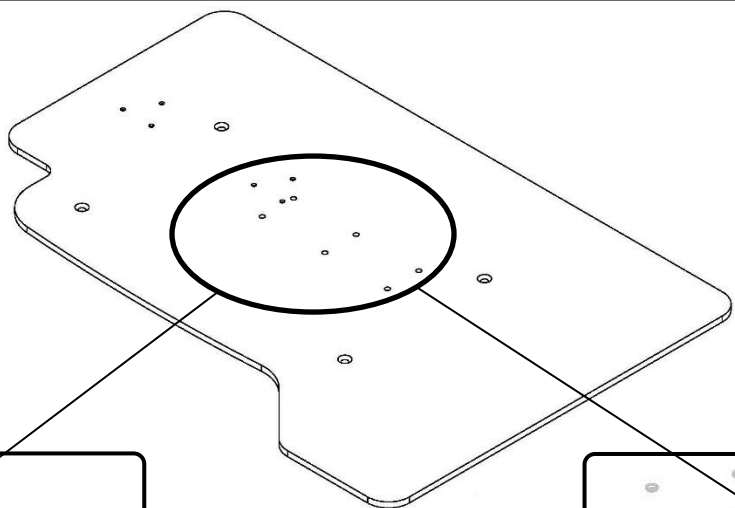


OR

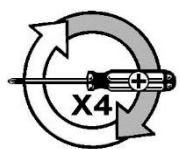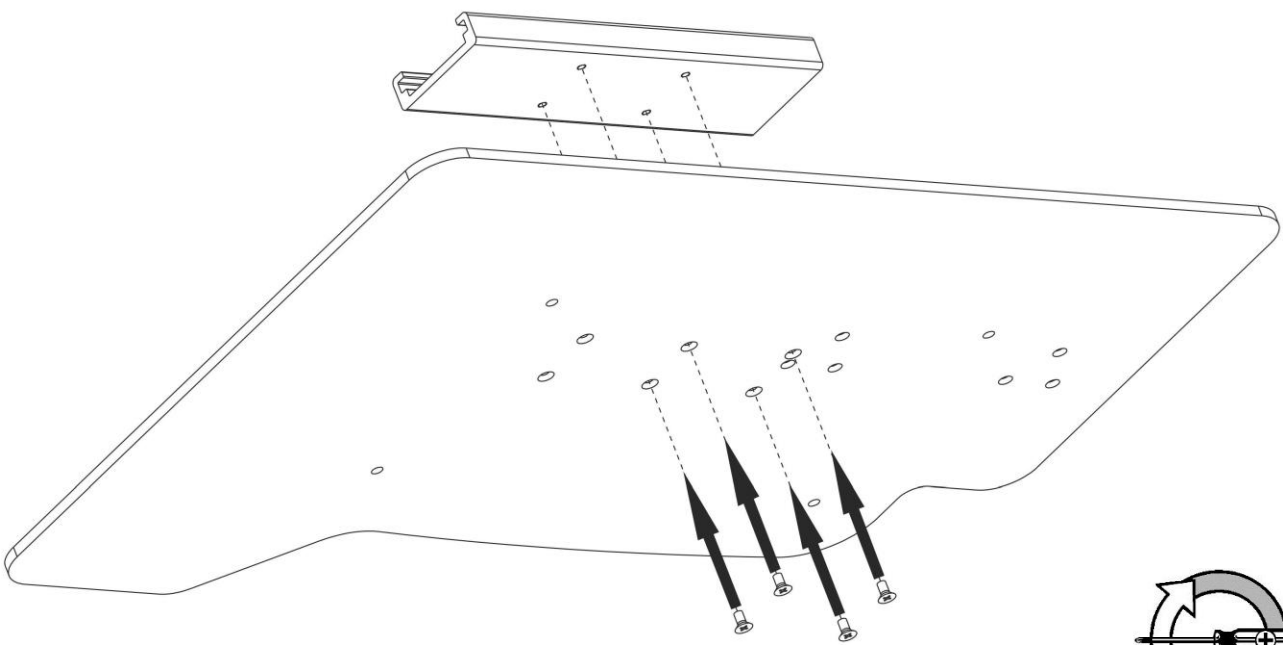


2




a

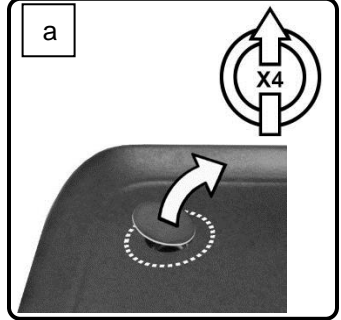

b

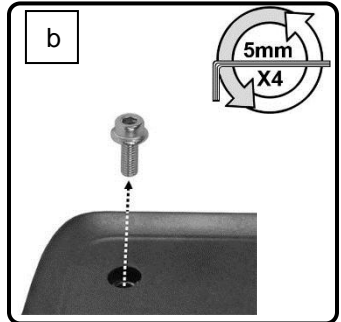


3


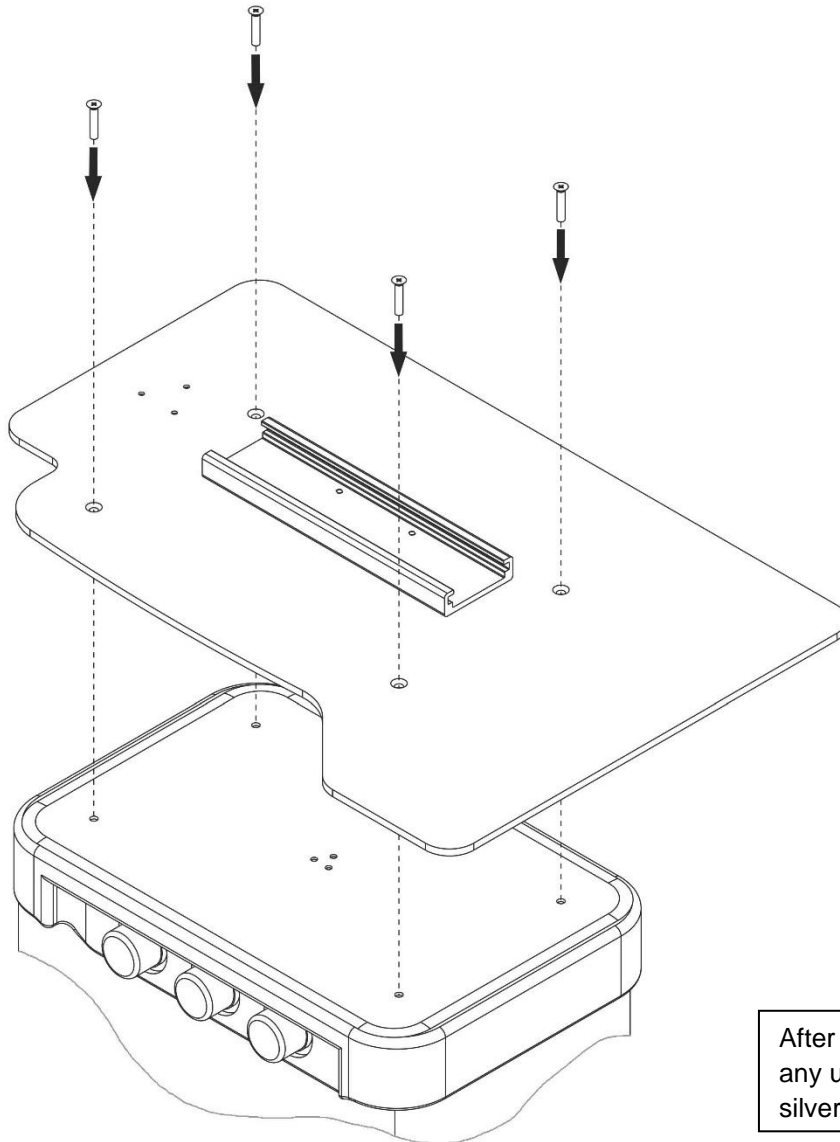


After installation, cover any unused holes with silver labels (supplied).


Carestation 600 Series Monitor Mount (Optional Accessory)

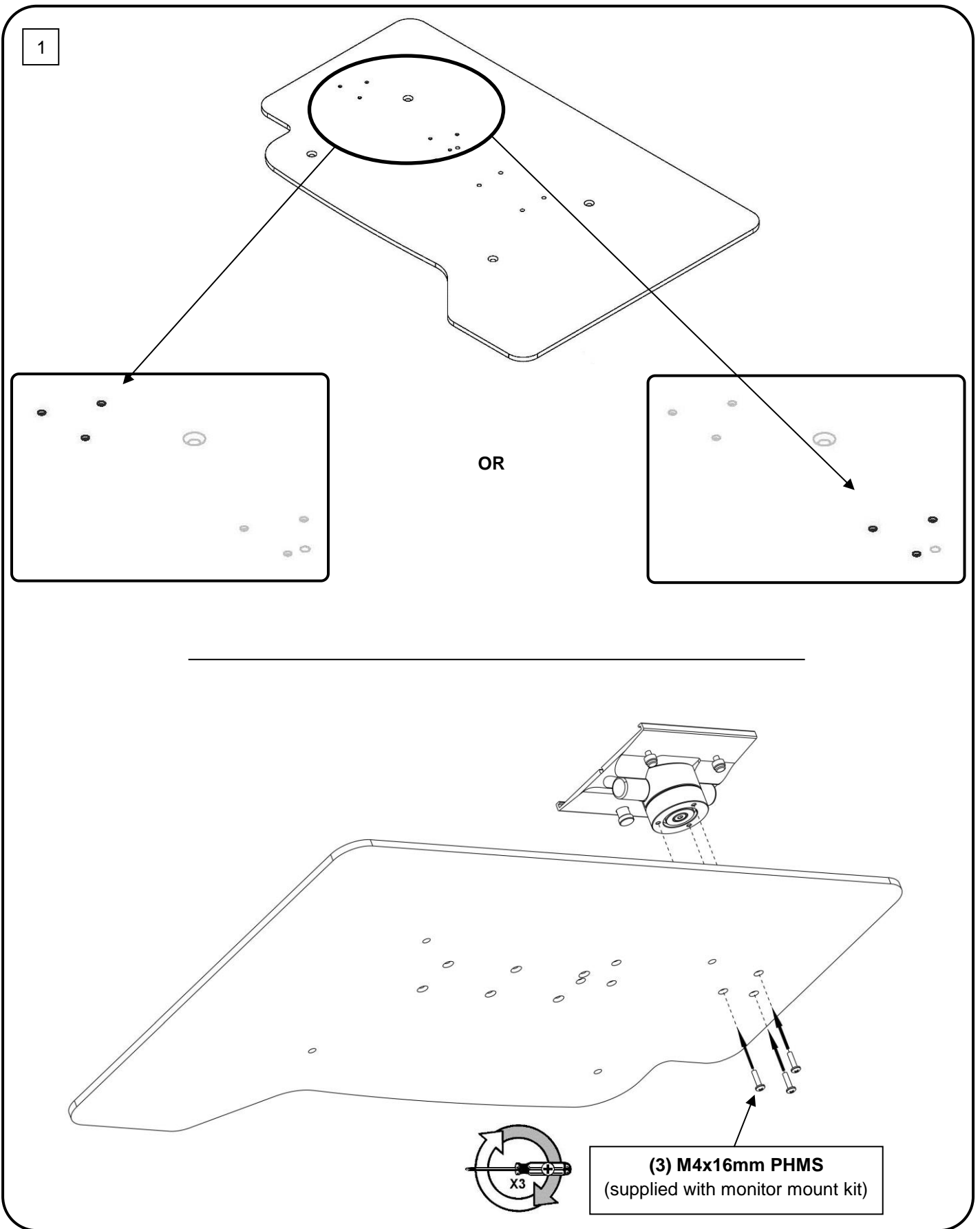

2

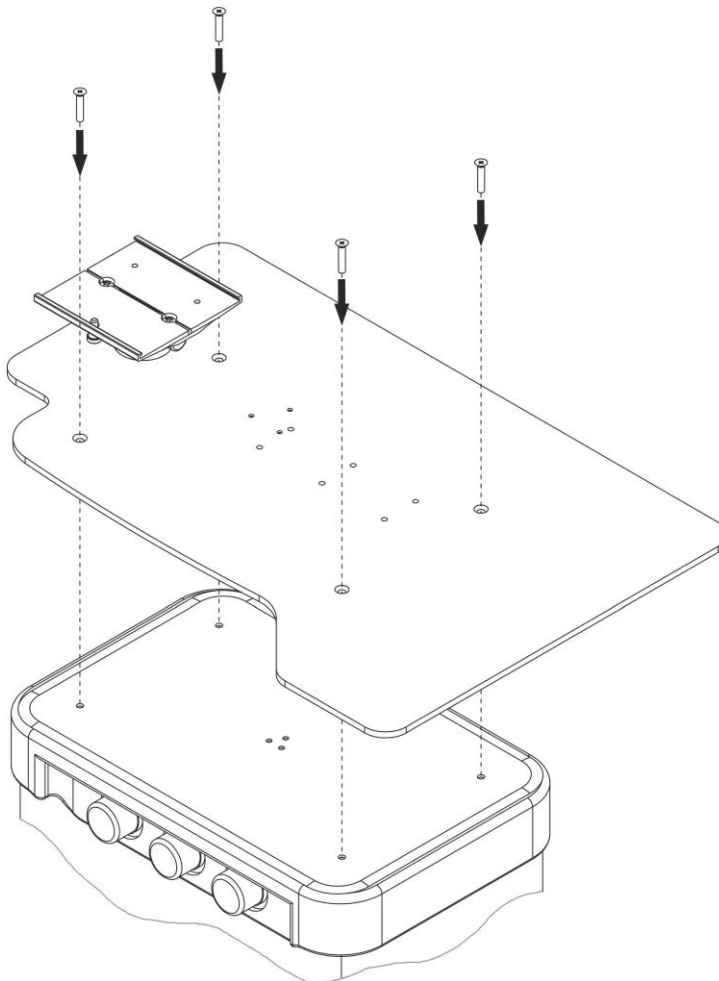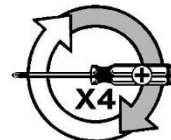


Tilt Adjustment

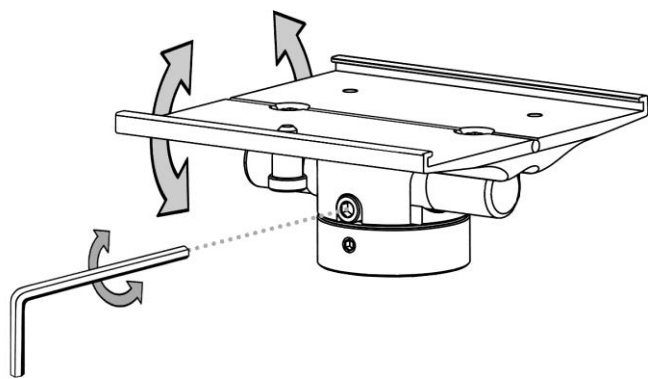

5mm

Swivel Adjustment

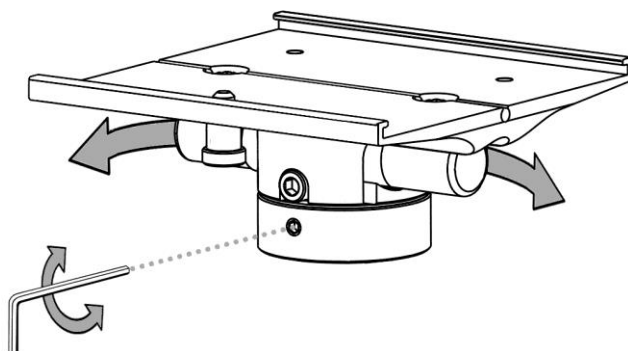

3mm

Routine Maintenance

Periodically inspect all fasteners associated with the mounting assembly. Tighten or Adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly


The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).

The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.

Steel wool or other abrasive material should never be used.

Damaged caused by the use of unapproved substances or processes will not be warranted. We recommend testing of any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.

Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off the mounting assembly immediately using a water-dampened cloth. Dry the assembly thoroughly after cleaning.

 **CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products.